QuickSplint®

Custom-fit Interim Bite Plane

Indications for Use Statement

- Protection of teeth and restorations from injury due to bruxism or clenching.
- Temporary relief of muscle tension, headaches and pain associated with bruxism or clenching.
- Temporary treatment of acute jaw joint and muscle sprain/strain along with the relief of associated headaches and pains.

Orofacial Therapeutics

Distributed by Orofacial Therapeutics, LP.
©2016 Orofacial Therapeutics, LP. QuickSplint® is a registered trademark of Orofacial Therapeutics, LP. All rights reserved. Made in USA

1-619-303-4606 | QuickSplint.com

PN D-3002RevG

See instructions for use
 Ver las instrucciones
 Voir les instructions pour l'utilisation
 Veja as instruções de utilização
 Vedere le istruzioni per l'uso
 Siehe Gebrauchsanweisung
 Zie de instructies voor het gebruik

Date of manufacture
 Fecha de fabricación
 Date de fabrication
 Data de fabrico
 Data di produzione
 Datum der Herstellung
 Datum van fabricage


Manufacture
 Fabricante
 Fabricant
 Fabricante
 Produttore
 Hersteller
 Fabrikant
 Truer Medical, Inc.
 An ISO 13485 Registered Co.
 1050 N. Batavia Ave. Suite C
 Orange, California 92867 USA

Use by
 Su uso per
 Utilisation par les
 Validade
 Da usare entro
 Die Nutzung durch
 Houdbaar to

Batch Code
 Código de lote
 Code de lot
 Código do lote
 Codice lotto
 Batch Code
 Batch Code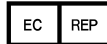

Advena Ltd.
 Pure Offices
 Plato Close,
 Warwick CV34 6WE UK

10° C - 24° C
 50° F - 75° F

EN	Prescriber Instructions	4
ES	Instrucciones para el especialista	8
FR	Instructions du prescripteur	22
PT	Instruções para o médico dentista	16
IT	Istruzioni per il medico prescrivente.....	20
DE	Anweisungen für den Verschreibenden....	24
NL	Instructies voor de voorschrijver.....	28

Orofacial Therapeutics

Distributed by Orofacial Therapeutics, LP.
 ©2016 Orofacial Therapeutics, LP. QuickSplint® is
 a registered trademark of Orofacial Therapeutics,
 LP. All rights reserved. Made in USA
1-619-303-4606 | QuickSplint.com

PN D-3002RevG

To Reorder go to:
QuickSplint.com
 or CALL 1-800-760-0526

QuickSplint®

Custom-fit Interim Bite Plane

Prescriber Instructions

Indications for Use Statement

- Protection of teeth and restorations from injury due to bruxism or clenching.
- Temporary relief of muscle tension, headaches and pain associated with bruxism or clenching.
- Temporary treatment of acute jaw joint and muscle sprain/strain along with the relief of associated headaches and pains.

Disclaimer

QuickSplint® may not be suitable for all patients. This product should be used with care in patients with jaw mobility restrictions or hypermobility.

Note

QuickSplint is to be used primarily when sleeping to prevent pain or damage to teeth or restorations resulting from extreme forces on the teeth with the activity known as grinding, bruxism or clenching. You may suggest the patient wear the QuickSplint during the day as well for pain relief or to promote healing from surgical procedures, under your direction.


Single Patient Use Only

Contraindications

- Known allergies to Lexan121 or Vinyl Polysiloxane (VPS).

Warnings

- QuickSplint should not be fit on or over compromised teeth.
- QuickSplint is not recommended for continuous wear (24 hours/day).
- The Vinyl Polysiloxane liner must engage the 3 retention features of the QuickSplint or you will need to reload and remake the QuickSplint.
- Ensure QuickSplint has a stable fit with adequate retention to prevent dislodging or loss during use.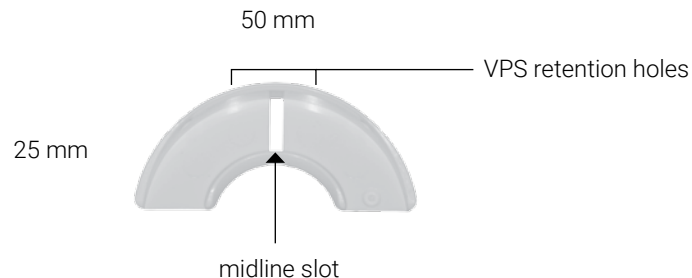

Overview

QuickSplint is designed to create a quick, ready-to-use custom appliance for transitional oral therapy. The patented features of QuickSplint allow fabrication without the use of adhesives. When filled with Vinyl Polysiloxane (VPS), the two overflow holes in the lip of the tray and the midline slot on the bottom of the tray will retain the VPS. VPS should overfill the entire tray, the midline slot and the overflow holes in order to achieve the best retention and fit to the patient's dentition.

Fast-set, heavy body VPS in cartridge or putty provide the best viscosity to ensure a secure fit. This material choice is critical to your experience with QuickSplint.

For additional information on how to make QuickSplint, see the instructional video on QuickSplint.com or the QuickSplint YouTube Channel, which contains tips and techniques.


YOU WILL NEED THE FOLLOWING

1. 7cc to 9cc of fast-set heavy body Vinyl Polysiloxane (VPS) bite registration material in putty or cartridge form, automix gun and mixing tips. Adhesives are not necessary. If you are using putty, you will need 4cc of the catalyst and 4cc of the base, to be mixed together by hand.
2. Protective gloves.
3. Sharp blade such as a scalpel for trimming.

WE PROVIDE THE FOLLOWING

1. QuickSplint in its own storage container.
2. Patient Instructions in the storage container.

To Reorder go to:
QuickSplint.com
 or CALL 1-800-760-0526

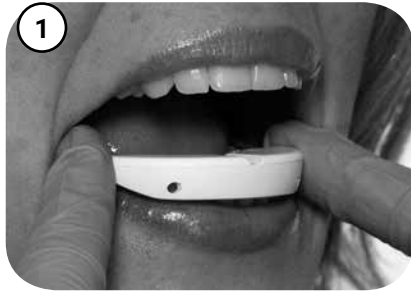**Step 1:
Determine Arch**

Determine which arch will provide maximum tooth contact on the QuickSplint surface or if the patient has a preference on whether QuickSplint shall be worn on the upper or lower arch. Confirm QuickSplint will fit the patient's arch by placing it in the patient's mouth and have the patient close to confirm.

**Step 2:
Fill Splint**

Overfill QuickSplint uniformly with VPS material. VPS should overflow the tray.

**Step 3:
Fill Midline Slot**

Turn the tray over and fill midline slot with VPS.

**Step 4:
Seat the Splint**

Align QuickSplint on the upper or lower teeth and support while the patient bites down completely to seat the QuickSplint. This levels the bite registration against the opposing arch. The VPS will overflow the QuickSplint, engaging the retention features and teeth. For upper arch, instruct patient to slowly and gently use tongue to position material against the back of the teeth and up on the palate or lingual area, to maximize retention.

**Step 5:
Remove Splint**

After the VPS sets, gently remove the QuickSplint.

**Step 6:
Trim Splint**

Trim any excess material with a sharp blade. Use the tray as your guide. Replace QuickSplint on patient's teeth and ensure there is a stable fit and retention to the teeth to avoid possible displacement with use. QuickSplint is now ready for use!

Cleaning instructions

1. Avoid hard brushing or other cleaning techniques which could tear or otherwise deform the VPS material.
2. Wash daily in warm water, rinse thoroughly and allow to air dry.
3. Periodically disinfect by soaking in half and half mixture of mouthwash and water for ten minutes.

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